

Vascushunt II Silicone Carotid Endarterectomy Shunt



Flexibility in shunting ... All the capabilities of Vascushunt carotid shunt, with a safety balloon and choice of sizes.




Edwards

The Vascushunt II carotid shunt is configured to minimize trauma to the vessel wall and allow easy access to high lesions.

Safety balloon


The incorporation of a safety balloon is designed to reduce the possibility of damaging the internal carotid artery. If the internal carotid balloon inadvertently overinflates, excess pressure is diverted to the safety balloon, reducing pressure in the internal balloon to help minimize the possibility of vessel trauma.*

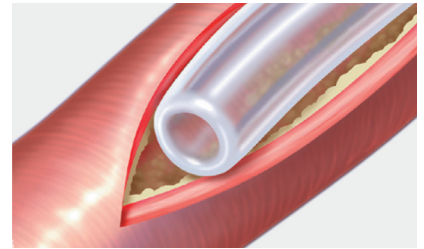

Silicone design to minimize trauma to the vessel wall.

Flexibility

The Vascushunt II silicone carotid endarterectomy shunt is designed to provide flexibility for ease-of-use during the procedure.

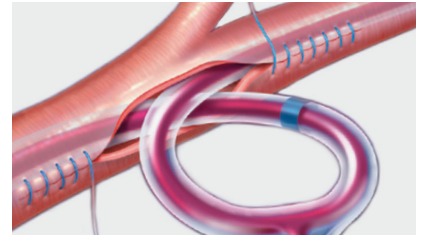

Silicone flexibility for ease-of-use during the procedure.

Multiple sizes and lengths

Available in 8F, 9F and 11F in both outlying 30 cm and inlying 15 cm lengths, with stopcocks to meet a variety of clinical needs.

*It has been reported that human internal carotid arteries can rupture at pressures of 20 psi. The safety balloon is intended to prevent the pressure from reaching this level by inflating at a level that is less than 20 psi.



Vascushunt II silicone endarterectomy shunt is not made with natural rubber latex.

Vascushunt II silicone balloon shunts

Specifications

Model number	Catheter french size	Length
T38F15	8F	15 cm
T38F30	8F	30 cm
T39F15	9F	15 cm
T39F30	9F	30 cm
T311F15	11F	15 cm
T311F30	11F	30 cm

CAUTION: Care must be taken at all times to prevent over-inflation of the balloons, as it may result in obstruction of blood flow and injury to the vessel.

CAUTION: Federal (United States) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

Edwards, Edwards Lifesciences, the stylized E logo, 1-800-FOGARTY, Fogarty, Vascushunt, and Vascushunt II are trademarks or service marks of Edwards Lifesciences Corporation or its affiliates. All other trademarks or service marks are the property of their respective owners.

© 2020 Edwards Lifesciences Corporation. All rights reserved. PP--US-1014 v2.0

Edwards Lifesciences • One Edwards Way, Irvine CA 92614 USA • edwards.com

Please contact your sales representative, visit Edwards.com/CarotidShunt or call 1-800-FOGARTY to learn more.

Manufactured by Lucas Medical, Inc.
Distributed by Edwards Lifesciences LLC.



Edwards